624

ART. 114.624

GranberG[®]**624**

ART. 114.624

GranberG[®]

PPE Cat. I

Available sizes: S-6/7, M-7/8, L-8/9, XL-9/10
 Tilgjengelige størrelser: S-6/7, M-7/8, L-8/9, XL-9/10
 Tillgängliga storlekar: S-6/7, M-7/8, L-8/9, XL-9/10
 Saatavilla olevat koot: S-6/7, M-7/8, L-8/9, XL-9/10
 Dostępne rozmiary: S-6/7, M-7/8, L-8/9, XL-9/10

For minimal risks only

EN: USER MANUAL**NO: BRUKERVEILEDNING****SV: BRUKSANVISNING****FI: KÄYTTÖOHJE****PL: INSTRUKCJA OBSŁUGI****GranberG**[®]**Head office:**

GRANBERG AS

Bjoavegen 1442, NO-5584 Bjoa, Norway
 Tel +47 53 77 53 00, e-mail post@granberg.no

Swedish office / European warehouse:

GRANBERG SVERIGE AB

Schubergvägen 20, 311 74 Falkenberg, Sweden
 Tel +46 (0)346 124 25, e-mail post@granberg-ab.se

granberggloves.com

User Manual issue date: 26.01.2023

ENGLISHDisposable Gloves Magic Touch[®]. Soft Nitrile[™], powder-free. Indigo colour.**EU Type-Examination**

This product is classed as Category I Personal Protective Equipment (PPE) according to European PPE Regulation (EU) 2016/425 and has been shown to comply with this Regulation through the Harmonised European Standard EN ISO 21420:2020.

Gloves comply with Regulation (EC) No. 1935/2004 and Regulation (EC) No. 2023/2006, and according to Commission Regulation (EU) No. 10/2011, they are suitable for handling all types of foods for short periods of time.

EU Declaration of Conformity is available on the following link:
<https://www.granberg.no/catalog/114.624>

Performance and Limitation of Use

It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.

Before usage, inspect the gloves for any defect or imperfections.

Components used in glove manufacturing may cause allergic reactions in some users. If allergic reactions occur, seek for medical advice immediately.

Product Instruction for Use

- I. Usage – For Single Use only. If re-used, the risk of contamination and infection increases due to improper cleaning processes; and increased risk of holes and tear during re-use due to weakening of gloves by cleaning processes.
- II. Sizing – Select the right size glove for your hand.
- III. Donning – Hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to a get a good fit. Don the other glove by the same procedure.
- IV. Inspection – Punctures or tears may occur after donning. Inspect each glove after donning, and immediately discontinue use if found damaged.
- V. During Use - Gloves polluted by blood or other organic fluids must be immediately removed as medical waste. In case of pricking or tearing, user should immediately change gloves.
- VI. In case of feeling looseness on fingers, user should change gloves.
- VII. Doffing – Hold glove bead and pull toward the finger until the glove come off.

Storage Conditions

Store in a cool and dry place in its original package. Opened boxes should be kept away from fluorescent and sunlight. Keep the gloves away from ozone, heating devices, and the source of the fire. Gloves are packed in a dispenser box suitable for transport. Keep the gloves in the box when not in use. The shelf life for products stored as recommended is mentioned on each package. Service life cannot be specified and depends on the application and responsibility of the user to determine the suitability of the glove for its intended use.

Further information can be obtained from the manufacturer, please contact Granberg AS.

NORSKEngangshansker Magic Touch[®]. Soft Nitrile[™], pudderfri. Indigo farge.**EC typegodkjenning**

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