612
ART. 114.612

GranberG®

612
ART. 114.612

GranberG®

Available sizes: XS, S, M, L, XL
Tilgjengelige størrelser: XS, S, M, L, XL
Tillgängliga storlekar: XS, S, M, L, XL
Saatavilla olevat koot: XS, S, M, L, XL
Dostępne rozmiary: XS, S, M, L, XL

GranberG®

Head office:
GRANBERG AS
Bjoavegen 1442, NO-5584 Bjoa, Norway
Tel +47 53 77 53 00, e-mail post@granberg.no

Swedish office / European warehouse:
GRANBERG SVERIGE AB
Schubergvägen 20, 311 74 Falkenberg, Sweden
Tel +46 (0)346 124 25, e-mail post@granberg-ab.se

granberggloves.com

EN: USER MANUAL
NO: BRUKERVEILEDNING
SE: BRUKSANVISNING
FI: KÄYTTÖOHJE
PL: INSTRUKCJA OBSŁUGI

ENGLISH

Single-Use Gloves Magic Touch®. Soft Nitrile™, powder-free. White colour.

EU Type-Examination

 Gloves are in compliance with Regulation (EU) No 10/2011 and are suitable for handling all kinds of foodstuffs for a short period of time.

Performance and Limitation of Use

It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.

Before usage, inspect the gloves for any defect or imperfections.

Components used in glove manufacturing may cause allergic reactions in some users. If allergic reactions occur, seek for medical advice immediately.

Product Instruction for Use

- I. Usage – For Single Use only. If re-used, the risk of holes and tear increases during re-use due to weakening of the gloves by cleaning processes.
- II. Sizing – Select the right size glove for your hand.
- III. Donning – Hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to get a good fit. Don the other glove by the same procedure.
- IV. Inspection – Punctures or tears may occur after donning. Inspect each glove after donning, and immediately discontinue use if found damaged.
- V. During Use - In case of pricking or tearing, user should immediately change gloves.
- VI. In case of feeling looseness on fingers, user should change gloves.
- VII. Doffing – Hold glove bead and pull toward the finger until the glove come off.

Storage Conditions

Store in a cool and dry place. Opened boxes should be kept away from fluorescent and sunlight.

Ideally stored in dry condition.

When stored as recommended, the gloves will not suffer any changes in chemical properties for up to three years from date of manufacture. Service life cannot be specified and depends on application and responsibility of user to ascertain suitability of the glove for its intended use.

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